
Nidek Medical Products, Inc Nuvo 8 Oxygen Concentrator Service Manual

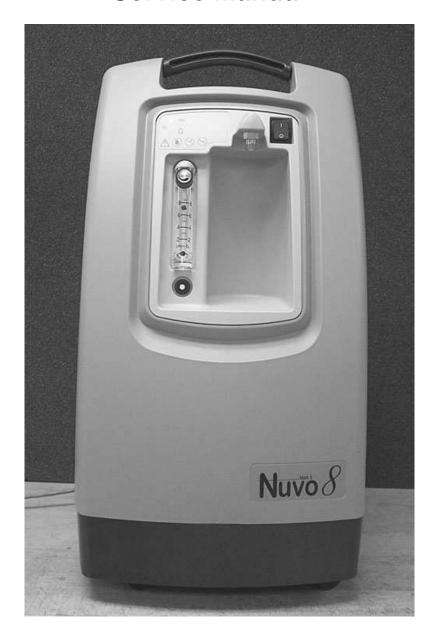

Nidek Medical Products, Inc. 3949 Valley East Industrial Drive Birmingham, Alabama 35217 USA Telephone: (205) 856-7200 • 24-Hour Fax: (205) 856-0533

Table of Contents		2-4
General Safety		
Instructions		
	Production and Use of Oxygen	5
	Use and Maintenance of the Device	5
	Standards and Regulations	5-6
Section 1.0	<u> </u>	
Introduction		
1.	Home Service Provider Responsibility	6-7
1.2	· · · · · · · · · · · · · · · · · · ·	7
1.3	, , , , , , , , , , , , , , , , , , , ,	8
Section 2.0	<u> </u>	
Operational Check and	Concentration Test	
2.		8
2.2	·	9
2.3	•	9
	2.3.1 Battery Test	10
	2.3.2 Power Failure Alarm test	10
2.4		10-11
Section 3.0	,	
Patient Instructions		
3.	General Instructions	11
3.2	Routine Maintenance by the Patient	11
	3.2.1 Cleaning the Cabinet Air Filter	11
	3.2.2 Checking the Alarm System Battery	12
Section 4.0		
Home Service Provide	r Maintenance	
4.′	Routine Maintenance	12
	4.1.1 Cabinet Air Filter	12
	4.1.2 Final Product Filter Replacement	12
	4.1.3 Inlet Air Filter Replacement	13
	4.1.4 Battery Replacement	13
	4.1.5 Recording Maintenance	13
4.2		13-14
	4.2.1 Preparing for New Patient Use	14
Section 5.0	-	
Service		
5.	1 Components	14-15
5.	Cabinet Removal	15
	5.2.1 Removing Cabinet Back	15
	5.2.2 Removing Cabinet Front	15
	5.2.3 Removing Control Panel	15
	5.2.4 Main Structure	15
	5.2.5 Caster Replacement	15

Page 2 of 52 2010-2229 Rev - B

	5.3	Compressor	15
		5.3.1 Compressor Replacement	15-16
		5.3.2 Capacitor Replacement	17
	5.4	Process Control Valve	17
	5.5	Sieve Module Replacement	18
		5.5.1 Sieve Module Removal	28
		5.5.2 Sieve Module Installation	18
	5.6	Cabinet Fan Replacement	19
	5.7	Circuit Board Replacement	19-20
		5.7.1 Circuit Board Removal	20
		5.7.2 Circuit Board Installation	20
		5.7.3 Timing Board Removal	20
		5.7.4 Timing Board Installation	21
	5.8	Product Regulator Check and Setting	21
		5.8.1 Product regulator cleaning or rebuilding	21
	5.9	Pressure Switch Replacement	22
	5.10	Circuit Breaker Replacement	22
		5.10.1 Circuit Breaker Removal	22
		5.10.2 Circuit Breaker Installation	22
	5.11	I/0 (ON/OFF) Power Switch Replacement	22
		5.11.1 I/0 (ON/OFF) Power Switch Removal	22
		5.11.2 I/0 (ON/OFF) Power Switch Installation	23
	5.12	Buzzer Replacement	23
	5.13	Hour Meter Replacement	23
	5.14	Flow Meter Replacement	23
		5.14.1 Flow Meter Removal	23
		5.14.2 Flow Meter Installation	23
	5.15	Power Cord Replacement	23-24
Section 6.0			
Troubleshooting			
	6.1	Air Pressure Test (P1)	24-25
		6.1.1 High Air Pressure	26
		6.1.2 Low Air Pressure	26
	6.2	Product Pressure Test (P2)	26-27
		6.2.1 High Product Pressure	27
		6.2.2 Low Product Pressure	27
	6.3	General Troubleshooting	27-29
	6.4	Troubleshooting Chart	30-32
	6.5	Tool Kit and Pressure Test Gauge	33
		<u> </u>	

2010-2229 Rev - B Page 3 of 52

Appendices

Exploded			
Drawings			
	A-1:	Pneumatic Flow Chart	34
	A-2:	Electrical Schematic, 230 Volt Unit	35
	A-3:	Electrical Schematic, 115 Volt Unit	36
	A-4:	Base Assembly	37
	A-5:	Main Structure Assembly Front View	38
	A-5.1:	Main Structure Assembly Rear View	39
	A-6:	Regulator Assembly	40
	A-7:	Compressor Assembly, 115 Volt Unit	41
	A-7.1:	Compressor Assembly, 230 Volt Unit	42
	A-8:	Control Panel Assembly; Front View	43
	A-8.1:	Control Panel Assembly; Rear View	44
	A-9:	Front Cabinet Assembly	45
	A-10:	Rear Cabinet Assembly	46
	A-11:	Module Assembly	47
	A-12:	Maintenance Log	48-49

Page 4 of 52 2010-2229 Rev - B

General Safety Instructions

Production and use of oxygen

Oxygen is not a flammable gas, but accelerates the combustion of materials. To prevent fire risks, the **Nuvo 8** should be kept away from flames, incandescent sources or sources of heat (cigarettes) and combustible products such as oil, grease, solvents, aerosols, etc.

Do not use in an explosive atmosphere.

Prevent oxygen from accumulating on upholstered seats or any other fabric. If the concentrator operates without being administered to a patient, locate it so that the flow of gas generated is dissipated into the air.

Locate the equipment in a free space (filter to the rear and below) which is well ventilated and free of fumes or atmospheric pollution.

Use and Maintenance of the Device

Use the electric cable provided and check that the voltage of the mains socket used complies with the electrical characteristics of the appliance indicated on the manufacturers plate on the rear of the appliance.

Do not use an extension cord or multiple sockets which can create sparks and therefore pose a fire risk.

Use of the **Nuvo 8** must be restricted solely to oxygen therapy on medical prescription in compliance with the daily rate and duration.

Use in other circumstances may represent a hazard to patient health.

Do not use in a specifically magnetic environment (MRI, etc.).

The Nuvo 8 has an audible alarm intended to warn the user of any problems. The user must determine the maximum distance away from the Nuvo 8, based on the sound levels in the environment, to ensure that the alarm is always audible.

Standards & Regulations

In compliance with UL60601-1 [EN60601-1] (para 6.82.b):

"The manufacturer, assembler, installer or importer are not considered to be responsible for consequences or the safety, reliability and characteristics of a device unless,

 the assembly, extensions, adjustments modifications or repairs have been performed by persons authorized by the manufacturer,

2010-2229 Rev - B Page 5 of 52

- the electrical installation of the corresponding premises complies with appropriate regulations and codes,
- the device is used in accordance with the instructions for its use.

If the replacement parts used for periodic servicing by an approved technician do not comply with the manufacturer's specifications, the manufacturer is absolved of all liability in the event of an incident.

Do not open the equipment when it is powered on: risk of electrocution.

This device complies with the requirements of the FDA Quality System Regulation and EU Directive 93/42/EEC, but its operation may be affected by the use in the surrounding area of appliances such as diathermy, high frequency electro-surgical instruments, defibrillators, short wave treatment appliances, cell-phones, CB devices and other portables, microwave ovens, induction hot plates or remote control toys, and more generally, by electromagnetic interference exceeding the levels specified in standard IEC(EN) 60601-1-2.

1.0 Introduction

1.1 Home Service Provider Responsibility

All Home Service Providers of the Nidek Medical Nuvo 8 Oxygen Concentrator must assume responsibilities for handling, operational check-out, patient instruction, and maintenance. These responsibilities are outlined below and throughout this manual.

WARNING

Nuvo 8 units must not be used for or with any life-supporting or life sustaining applications. Patients unable to communicate discomfort while using this device may require additional monitoring. Advise patients to immediately notify their Home Service Provider(s) and/or physician(s) in case of an alarm or any discomfort.

As a Home Service Provider, you must do all of the following:

- Inspect the condition of each Nuvo 8 unit immediately upon delivery to your business location. Note any sign of damage, external or internal, on the delivery receipt, and report it directly to both the freight company and Nidek Medical Products, Inc. immediately.
- Check the operation of each Nuvo 8 before delivery to a patient. Always operate the unit for a reasonable length of time and check that the oxygen concentration level is within specifications as referred to in Section 2.4. Test the battery and power disconnect alarm as described in Section 2.3 of this manual.
- Deliver Nuvo 8 units only to patients authorized by a physician's prescription. The Nuvo 8 must not be used as a life-supporting or life sustaining device. A backup supply of oxygen must be available.

Page 6 of 52 2010-2229 Rev - B

- Instruct patients how to use the Nuvo 8 in conjunction with the Patient Manual.
- Instruct patients to notify their physicians and/or Home Service Providers if they experience any signs of discomfort.
- Instruct each patient how to perform routine maintenance of the cabinet air filter and how to check the alarm system battery, if equipped. (Refer to Section 3.2.)

Be available to service each patient at any time. Maintain the **Nuvo 8** in accordance with Section 4.0.

Repair components and replace parts only as outlined in this manual. Use only Nidek Medical parts for replacement in Nuvo 8 Oxygen Concentrators.

 Refer to the Nuvo 8 Product Warranty if parts replacement is required within the warranty period.

1.2 Important Notice and Symbol Explanations

As you read the manual, pay special attention to the WARNING, CAUTION, and NOTE messages. They identify safety guidelines or other important information as follows:

WARNING:	Describes a hazard or unsafe practice that can result in severe bodily injury or death.
CAUTION:	Describes a hazard or unsafe practice that can result in minor bodily injury or property damage.
NOTE:	Provides information important enough to emphasize or repeat.

The following harmonized symbols (pictograms), used for non-English language countries, will be located on the exterior of the **Nuvo 8** unit:



Read the accompanying documents; particularly the patient operating guide.



Store, ship and use the device in an upright condition.



No smoking within five feet of this device, oxygen-carrying tubing, or accessories.



Indicates an alarm signal.



Do not use any oil or grease on or near the device

2010-2229 Rev - B Page 7 of 52

1.3 Functional Specifications

Dimensions: 15.5 in. long, 15.6 in. wide, 27.8 in. tall

(394 mm long, 396 mm wide, 706 mm high)

Weight: 50 to 54 lb (22.5 to 24.5 kg)

Electrical 120 VAC, 60 Hz, 4.4-5.0 amps, 500 watts avg Requirements: 230 VAC, 50 Hz, 1.8-2.1 amps, 490 watts avg

230 VAC, 60 Hz, 2.4-2.7 amps, 585 watts avg

Capacity: 8 liters per minute at 90% oxygen

(Based on 70°F [21°C] at sea level)

Accuracy: Flow meter ±5% full scale. (Based on pre-calibrated

analysis.)

Concentration: 2-8 liters per minute at 90% + 6.5 / - 3%

Response Time: Acceptable concentration is normally achieved in about

90 seconds; allow 5 minutes to attain full concentration.

Positioning: Operate the unit in an upright position, maintaining at

least six inches of open space on all sides for ventilation.

2.0 Operational Check and Concentration Test

2.1 Description of Operation

Air enters the **Nuvo 8** Oxygen Concentrator through an external cooling air intake filter. This filtered air enters the compressor via a suction resonator and fine filter, which quiets the suction sounds made by the compressor. Pressurized air then exits the compressor and passes through a heat exchanger, which reduces the temperature of the compressed air. Next, an electronic valve system directs the air into one of two sieve beds that contain molecular sieve. The special characteristic property of molecular sieve is that it physically attracts (adsorbs) nitrogen when air passes through this material, thus enabling the production of high purity oxygen.

There are two sieve beds or adsorbent columns; while one produces high purity oxygen, the other is purged of the nitrogen it adsorbed (collected) while it was producing oxygen. Each column produces oxygen for approximately 5.5 seconds and delivers it to the product storage volume in the end of the column. Oxygen exits the adsorbent column through a pressure regulator, flow control valve, flow meter, and final product filter. The flow control valve, which is part of the flow meter, controls the flow rate of oxygen delivered to the patient. The Nuvo 8 unit delivers up to 95% oxygen concentration at flow rates from 2 to 8 l/min. The remaining constituents of the product gas stream are nitrogen and argon, both of which are part of the air we breathe, are inert and are completely safe.

Page 8 of 52 2010-2229 Rev - B

2.2 Operational Check

Nidek Medical runs each device through a burn in period and tests every Nuvo 8 Oxygen Concentrator thoroughly after manufacture before releasing for shipment. As the home service provider, it is your responsibility to perform the following test to ensure that no damage occurred in shipping or handling.

- Open and inspect all concentrator cartons upon receipt. Unpack each unit and remove it from its carton. Inspect the unit itself for damage. If the exterior of the carton is damaged, or the unit itself is damaged, note it on the freight bill signed by the driver.
 - Before plugging the unit into the wall outlet, actuate the I/0 (ON/OFF) switch to the I (ON) position and note that the audible alarm sounds continuously. (See Section 2.3). Move the switch to the 0 (OFF) position.
- 2. Plug in the power cord of the unit, and set the I/0 (ON/OFF) switch to the I (ON) position. Check to see that the following occurs:
 - The compressor runs, listen for the sound.
 - Exhaust air flows out of the bottom of the unit.
 - OPTIONAL for Units Equipped with Oxygen Concentration Status Indicator (OCSI): The OCSI green light remains off until the oxygen concentration reaches 85% @ 60 Hz and 83% @ 50 Hz. (approximately two minutes).
- Turn the flow meter adjustment knob counterclockwise until it stops (wide open). The flow meter should register in excess of 8.5 liters/min. If not, refer to Section 5.8 to adjust the product regulator.
- 4 Perform an oxygen concentration test, as described in Section 2.4.

2.3 Alarm System

The Nuvo 8 Oxygen Concentrator is equipped with an alarm system, which sounds a continuous and loud alarm when a power failure occurs or when one or more cycle variables are not within specification. It sounds an alarm if the high or low pressure indicators are activated (standard models only) or if the optional OCSI detects lower than predetermined levels of oxygen concentration. The alarm remains on until you correct the alarm condition or you set the I/O (ON/OFF) switch to the 0 (OFF) position. Refer to Section 6.0 for a list of probable alarm causes.

2010-2229 Rev - B Page 9 of 52

2.3.1 Battery Test

If equipped, the Nuvo 8 battery should be checked by observing that the continuous alarm sounds when the I/O (ON/OFF) switch is moved to the I (ON) position when the unit is unplugged from the wall outlet.

2.3.2 Power Failure Alarm Test

To test the power failure alarm, perform the following actions:

Unplug the power cord from the wall outlet, and set the I/0 (ON/OFF) switch to the I (ON) position.

This should immediately activate the audible alarm. If it doesn't, refer to the troubleshooting chart in Section 6.0 of this manual.

2.4 Oxygen Concentration Test and Specification

To ensure that the output of oxygen from the device is within specification, you must perform an oxygen concentration test. Test the unit upon delivery to a patient and at periodic intervals. Home Service Providers, based on their expertise and documentation, may establish and implement their own plans for checking oxygen concentration. Consult Nidek Medical's Service and Maintenance Log (A-12) for the recommended intervals for testing.

- 1. If an oxygen humidifier bottle is used, remove it from the oxygen outlet.
- 2. Connect a calibrated oxygen concentration analyzer to the oxygen outlet.
- 3. Set the I/O (ON/OFF) power switch to the I (ON) position. (It takes approximately five minutes for the oxygen concentration to stabilize.) Take oxygen concentration readings over a period of several minutes to reduce any cyclic variations
- 4. Verify that the product flow rate delivered by the unit matches the patient's prescription and does not exceed the capacity of the unit.
- 5. Disconnect the oxygen analyzer, and reconnect the humidifier bottle (if used) and any other equipment / accessories that may be required.
- 6. Adjust the flow meter adjustment knob to the prescribed flow rate.

NOTE:

Do not measure oxygen concentration output after the product stream passes through a humidifier bottle, erroneous readings will result and your oxygen concentration measuring device might be damaged.

Page 10 of 52 2010-2229 Rev - B

Nidek Medical Nuvo 8 Concentration Specifications

Liter Flow	<u>Specification</u>	Within Specification
2 to 8 l/min	90% + 6.5 / - 3%	87% or higher

3.0 Patient Instructions

3.1 General Instructions

It is important that patients thoroughly understand how to operate the Nidek Medical Nuvo 8 unit. This enables proper treatment as prescribed by a qualified, licensed physician. You must explain that the purpose of this therapy is to alleviate symptoms. If patients experience any discomfort or the unit alarms, they must notify their Home Service Provider and/or physician immediately. You, as the Home Service Provider, are responsible to see that each patient receives the Patient Manual. Explain each step in the operation of the unit to the patient in reference to this manual.

3.2 Routine Maintenance by the Patient

To ensure accurate output and efficient operation of the unit, the patient must perform two simple routine maintenance tasks:

- Clean the cabinet air filter
- Check the alarm system

3.2.1 Cleaning the Cabinet Air Filter

NOTE:

The patient must clean this filter weekly, as described below. The filter may require daily cleaning if the Nuvo 8 unit operates in a harsh environment such as a house heated by wood, kerosene, or oil, or one with excessive cigarette smoke.

- 1 Remove the dirty cabinet air filter from the back of the Nuvo 8 unit.
- 2 Wash the dirty filter in warm water with household detergent, and rinse.
- 3 Use a soft absorbent towel to remove excess water.
- 4 Reinstall the clean cabinet air filter on the grille in the back of the unit.

3.2.2 Checking the Alarm System Battery

See Procedure described in Paragraph 2.3.1

4.0 Home Service Provider Maintenance

4.1 Routine Maintenance

The Nuvo 8 unit has two filters and a 9-volt battery, if equipped, that require scheduled maintenance and replacement.

To ensure that the output of oxygen from the unit is within specification, you must perform an oxygen concentration test. Test the unit upon delivery to a patient and at periodic intervals. Home Service Providers, based on their expertise and documentation, may establish and implement their own practices for checking oxygen concentration. The interval established may be longer or shorter than 90 days, which is the default time period recommended for providers who do not choose to establish their method.

Nidek Medical does not require preventive maintenance on the concentrator. You do not need to perform any maintenance as long as the Nuvo 8 unit remains within specifications at the prescribed flow rate. (Refer to Section 2.4)

4.1.1 Cabinet Air Filter

The external cabinet air filter is located on the back of the unit; it is easily removed by hand. Instruct the patient to clean this filter weekly. (Refer to Section 3.2.1.)

NOTE:

The filter may require more frequent cleaning if the Nuvo 8 unit operates in a harsh environment such as a house heated by wood, kerosene, or oil, or one with excessive cooking, cigarette smoke or atmospheric dust.

4.1.2 Final Product Filter Replacement

The final product filter does not require periodic replacement; it needs to be replaced only if it restricts oxygen flow. It is suggested that it be replaced whenever the sieve module is repaired or replaced and after the compressor is rebuilt.

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the cabinet back to locate the final product filter. **NOTE:** Observe the flow direction of the filter before removal.
- 3. Separate the silicone tubing from both sides of the filter.
- 4. Install the new filter with the inlet side in the same position as before. Push the tubing together so that it overlaps the barbs of the final product filter connections.
- Record information about the final product filter replacement in Appendix 12 of this manual and online at www.nidekmedical.com under the 'Maintenance Log' tab.
- 6. Reinstall the cabinet back.

Page 12 of 52 2010-2229 Rev - B

4.1.3 Inlet Air Filter Replacement

The inlet air filter requires inspection at each patient visit. The filter should be replaced annually, or more often depending on the environment.

- 1. Set the unit I/0 (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the rear cabinet access panel to locate the intake filter.
- 3. Remove filter from the unit, and replace with a new filter.
- Record information about the filter replacement in Appendix 12 of this manual and online at www.nidekmedical.com under the 'Maintenance Log' tab.
- 5. Reinstall the access panel.

4.1.4 Battery Replacement

Each time the Nuvo 8 unit is turned on, the alarm must sound loudly for approximately five seconds to indicate a good battery. An alarm that does anything other than sound loudly for five seconds indicates a weak battery and requires replacement. For units with OCSI, actuate the switch with the power cord unplugged to test the battery. To replace the battery, take the following steps:

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the access panel in the cabinet back.
- 3. Pull the battery from the mounting slot and remove the connection wires.
- 4. Attach the new battery to the connector and reinsert it into the mounting slot.
- 5. Set the I/O (ON/OFF) switch to the I (ON) position to test the alarm.
- 6. Record the battery replacement information in Appendix 12 of this manual and online at www.nidekmedical.com under the 'Maintenance Log' tab.
- 7. Reinstall the access panel.

4.1.5 Recording Maintenance

As the Home Service Provider, it is suggested that you record all routine maintenance and repairs performed on the Nuvo 8 unit, including hours and dates of service in Appendix 12 of this manual and online at www.nidekmedical.com under the 'Maintenance Log' tab.

4.2 Cleaning and Infection Control

With the growing concern about possible cross infection from home oxygen equipment (i.e., oxygen concentrators) from one home care patient to another, appropriate cautions should be exercised. The organisms of most concern are M. Tuberculosis, HIV, and Viral Hepatitis. These are potentially pathogenic.

Tuberculosis can survive outside of the human body, but its mode of transmission is by droplet nuclei. When infected individuals cough, they release droplet nuclei into 2010-2229 Rev - B

Page 13 of 52

the air, and these carry the Tuberculosis organism. Another person may breathe in these droplet nuclei, but prolonged exposure to the infected person is usually necessary for infection to occur.

HIV and Viral Hepatitis are both viruses, which are not living cells themselves but which can duplicate when in a living "host" cell. Both of these organisms are usually passed on by person-to-person contact, and both need to be in the human body to survive. Once outside the body, viruses can survive for only a short period of time.

4.2.1 Preparing for New Patient Use

When you remove the Nuvo 8 from a patient's home, always dispose of the used nasal cannula and humidifier bottle. Clean the exterior of the Nuvo 8 with a soapy water solution or commercial detergent cleaner to remove any debris, organic or otherwise. Be careful not to get any liquid into the interior of the unit.

Next, clean the exterior with either a common chemical disinfectant or a bleach solution* and allow it to air dry. For the bleach solution, wear eye and skin protection to prevent exposure to the chlorine. Retest the Nuvo 8 before you return it to your inventory.

Replace the cabinet air filter between each patient's use or clean with warm soapy water if it is in good condition. Clean this filter at least once per week or more frequently if operated in a dusty environment.

*Make the bleach solution a 1:100 dilution of 5.25% sodium hypochlorite. Mix one part household bleach (e.g., Clorox) with 99 parts cold tap water. To measure the solution easily, take 1/4 cup of household bleach, and mix it with a gallon of cold tap water. Allow the mixture to sit on potentially contaminated surfaces for 10 minutes.

5.0 Service

5.1 Components

The design of the Nidek Medical Nuvo 8 Oxygen Concentrator allows for easy access and removal of most components. This allows you to perform scheduled maintenance, repair, and replacement of parts with minimal time and effort. The inlet air filter and battery are conveniently located inside the small access panel in the cabinet back.

CAUTION:

For your safety, be sure to set the I/O (ON/OFF) switch to the O (OFF) position and unplug the power cord before you service the Nuvo 8 Oxygen Concentrator.

NOTE:

Record all scheduled maintenance. (Refer to Section 4.1.5.)

5.2 Cabinet Removal

5.2.1 Removing Cabinet Back

To remove the cabinet back, remove two screws near the base (one on each side) and two inset screws at the top just under the handle.

5.2.2 Removing Cabinet Front

To remove the cabinet front, remove two screws near the base (one on each side) adjacent to the rear cabinet screws and two inset screws at the top just under the handle. It is necessary to remove the cabinet back to access the top screws.

5.2.3 Removing Control Panel

Eight screws hold the control panel in place; the front cover must be removed to access these screws. It is necessary to remove the electrical wiring and pneumatic tubing to fully release the control panel.

5.2.4 Main Structure

The weight and forces of the internal components rest solely on four parts: the main structure, the compressor plate, the base-plate and the base. These parts were specially designed and formed. They should never require replacement under normal use.

5.2.5 Caster Replacement

The casters are a push in type that do not require any fasteners. Lay the device on its back to access the casters from the bottom. Pull them straight out away from the bottom.

5.3 Compressor

The compressor is the pump within the oxygen concentrator that supplies air to the separation process performed by the sieve beds. The pressure generated by the compressor forces oxygen to flow out of the top of the sieve columns.

The compressor is the likely cause of two potential specific problems:

- a. An insufficient amount of air is supplied to the process.
- b. An excessive sound level.


Air Supply

Compressor output refers to how much compressed air the compressor can produce. This depends upon the model of the compressor, length of stroke, piston diameter, speed of rotation and condition of seals. The cup seals form the seal between the piston and the cylinder wall. As the cup seals wear, the output begins

to gradually decrease. This reduction in compressor output results in less air, and thus less oxygen, entering the sieve beds. Therefore, the production of oxygen decreases.

Because this drop in oxygen production occurs over a long period of time, preventive maintenance on the compressor is not required.

You can continue a patient's therapy on the Nuvo 8 unit as long as the oxygen concentration level at the prescribed liter flow rate is within Nidek Medical's specification limits. Refer to Section 2.4.

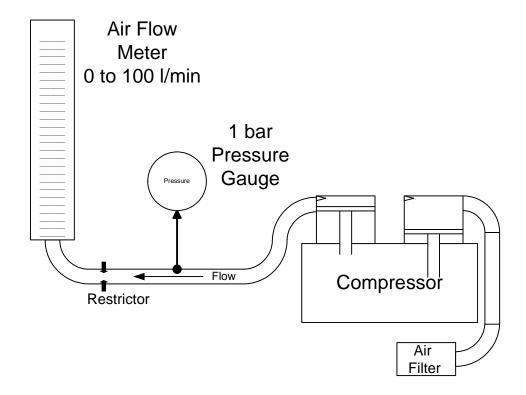

Sound Level

The sound level is largely determined by the condition of the compressor's bearings.

There are four bearings located within the compressor that allow the inner components of the compressor to rotate. If the bearings wear to the point that they become loose and noisy, the compressor becomes noticeably loud and needs servicing. The life of a compressor is determined primarily by its operating temperature. It is extremely important that the inlet cooling air filters are cleaned and replaced as required.

Page 16 of 52 2010-2229 Rev - B

5.3.1 Compressor Replacement

Remove Compressor Assembly

To remove the compressor assembly for exchange, follow the steps listed below:

- 1. Set the unit's I/0 (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- Remove the cabinet back and front.
- Disconnect the suction tube.
- 4. Disconnect the two compressor power cable leads and the two leads to the capacitor at the inline connector.
- 5. Remove the three screws that connect the compressor plate to the base of the unit, lift and slide out the compressor assembly.
- 6. Remove tie wrap from compressor plate wiring harness.
- 7. Remove compressor from the plate by removing the four compressor bolts.
- 8. Remove the intake and exhaust fittings.

Compressor Assembly Installation

To install a new or rebuilt compressor assembly, follow the steps listed below:

- 1. Perform the compressor removal procedure in reverse order.
- Leak test all connections.

5.3.2 Capacitor Replacement

The capacitor helps the compressor to start and run more efficiently. If the compressor cannot start, the capacitor may be defective and require replacement. The capacitor should be replaced at each compressor service / module replacement. To replace the capacitor, take the following steps:

- 1. Set the unit's I/0 (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the rear and front cabinets.
- 3. Disconnect the two leads to the capacitor and remove the nut from the mounting screw.
- 4. Remove the nut and washer below the capacitor.
- 5. To install the new capacitor, connect the leads and replace the capacitor and associated hardware.

2010-2229 Rev - B Page 17 of 52

5.4 Process Control Valve

The Nuvo 8 uses an electronic valve system valve assembly to control the air separation process. There is a feed port that connects to the heat exchanger and an exhaust port that connects to the exhaust muffler outlet. There are three possible valve states as follows:

- 1. Air feed connected to sieve bed A and exhaust connected to sieve bed B.
- 2. Air feed connected to sieve bed B and exhaust connected to sieve bed A.
- 3. Air feed connected to sieve bed A & B, both exhaust closed (normal deenergized condition).

The control valve of the Nuvo 8 unit requires no scheduled maintenance. If a valve does not function as required, it is best to replace the complete sieve module as it is probable that one or both of the beds has been damaged.

5.5 Sieve Bed/Module Replacement

CAUTION:

Do not expose molecular sieve (contents of bed) to air for an extended period of time. Prolonged exposure of molecular sieve to the moisture in room air results in contamination and permanent damage to the sieve material. Keep all openings to the sieve beds sealed during periods of storage.

NOTE:

If replacement is necessary, you must replace the entire module at the same time. It is recommended to replace the sieve bed and control valve module as a complete assembly.

5.5.1 Sieve Bed/Module Removal

- 1. Set the unit's I/0 (ON/OFF) switch to the 0 (OFF) position and unplug the power cord.
- Remove the cabinet back.
- 3. Disconnect the 3/8-inch air inlet tube from the side of the valve body.
- 4. Disconnect the 1/4-inch product outlet tube from the top of valve.
- 5. Disconnect the electrical wiring from each solenoid valve, noting there color and position.
- 6. Remove the tie-wrap (24" from around module).
- 7. Remove "Z" bracket; from the top of the module.
- 8. Lift the module up and out of the base plate.

Page 18 of 52 2010-2229 Rev - B

5.5.2 Sieve Bed/Module Installation

To install the sieve beds, follow the sieve bed/module removal procedure in reverse order. It is very important to properly connect all tubes to eliminate leaks.

To check for leaks, take the following steps.

- 1. Plug in the unit.
- 2. Set the unit's I/O (ON/OFF) switch to I (ON) for three minutes with the flow meter closed to pressurize the system.
- 3. Apply soapy water around all of the hose connections at the valve and check for leaks.

CAUTION:

There is an electrical shock hazard with the Power ON. Be careful that no water contacts any of the electrical connections or components.

NOTE:

Small leaks can affect a concentrator's performance and can cause contamination of the sieve. Careful leak testing is important.

5.6 Cabinet Fan Replacement

The cabinet fan for the Nuvo 8 is located under the compressor. Refer to the troubleshooting chart in Section 6.0 of this manual for instances where replacement of the fan may be required.

To replace the cabinet fan in the Nuvo 8 unit, take the following steps:

- 1. Set the unit's I/0 (ON/OFF) switch to the 0 (OFF) position and unplug the power cord.
- 2. Remove the front and back cabinets.
- 3. Follow the procedure in 5.3.1 to remove the compressor assembly.
- 4. Disconnect the fan leads.
- 5. Position the new cabinet fan so that the air flow arrow points toward the bottom and the electrical connections. Be sure that the electrical wires do not interfere with the rotation of the fan.
- 6. Connect the fan leads and place the fan on the mounting pins.
- 7. Replace compressor assembly.
- 8. Reinstall the front and back cabinets.

5.7 Circuit Board Replacement

The printed circuit board controls the alarm system functions.

Consult the troubleshooting chart in Section 6.0 to determine when to replace the printed circuit board.

CAUTION:

The Printed Circuit Board (PCB) contains components that are sensitive to electrostatic discharge (ESD) that can damage the board if not handled properly. As when handling any ESD sensitive PCB, observe standard ESD safety procedures. These procedures include the following:

- Handle the PCB by the edges only.
- Work on a grounded ESD mat.
- Wear a grounded wrist strap.
- Store PCB in anti-static bags only.

5.7.1 Circuit Board Removal (Note that the Circuit Board on the OCSI unit is different to the Circuit Board on the standard unit.)

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power 1cord.
- 2. Remove the front and cabinet backs.
- 3. Remove the control panel to which the circuit board is attached.
- 4. Disconnect the 7-pin connector from the circuit board.
- 5. Disconnect tubing from each end of the black sensor tube on the OCSI unit, noting their position and orientation.
- 6. **Non OCSI units only:** Cut tie-wrap and remove pressure sensor line.
- 7. Remove three screws that attach the board to the control panel.
- 8. Remove the circuit board.

NOTE:

Handle the new circuit board only by the edges to prevent electrostatic damage to the unit.

5.7.2 Circuit Board Installation

- 1. Locate the circuit board to line up the attachment points.
- 2. Install the three mounting screws.
- 3. Reinstall the tubing to each end of the black sensor tube as it was before.
- 4. **Non OCSI unit only**: Reinstall the pressure sensor line and tie-wrap.
- 5. Plug in the 7-pin connector.
- 6. Reinstall the control panel and the front and cabinet backs.

5.7.3 Timing Board Removal

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- Remove the cabinet back.
- 3. Disconnect the 6 and the 4 pin connectors as well as the 6 spade connectors from the Timing Board.
- 4. Remove the mounting screws.
- Remove the circuit board.

Page 20 of 52 2010-2229 Rev - B

NOTE:

Handle the new circuit board only by the edges to prevent electrostatic damage to the unit.

5.7.4 To reinstall the timing board, reverse the removal procedure.

5.8 Product Regulator Check and Setting

The product regulator enables you to set the maximum flow of oxygen output by the Nuvo 8 unit. To check for proper adjustment of the product regulator, take the following steps:

- 1 Set the I/0 (ON/OFF) switch to the I (ON) position.
- 2 Allow the unit to run for a few minutes.
- Remove the front and cabinet backs.
- 4. Connect a pressure gauge directly to the patient outlet.
- 5. The pressure should read $17 \pm 10\%$ psig ($117 \pm 10\%$ kPa).
- 6. Adjust the regulator if necessary. Turn the knob clockwise to increase the output pressure (requires a 5/32 hex wrench).
- 7. Reinstall the cabinet front and cabinet backs.

5.8.1 Product Regulator Cleaning or Rebuilding

Clean or rebuild the product regulator if the flow meter ball fluctuates more than 1/4 of a liter or if the regulator cannot be adjusted.

- I. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord,
- Remove the front and rear cabinets.
- 3. Cut the tie wrap holding the regulator in place. Remove from the main structure.
- 4. Adjust the product regulator fully counterclockwise to unload the spring. This makes disassembly and reassembly easier.
- 5. Remove the diaphragm. (Clean or replace it.)
- 6. Use a hex-head screwdriver to unscrew the diaphragm stem guide located in the center of the regulator body to gain access to the seat,
- 7. Remove the seat. Be careful not to lose the spring located behind the seat.
- 8. Replace the seat or clean by blowing clean air on and around it.
- 9. With the spring behind the seat, screw the diaphragm stem guide back into the body of the regulator. (Do not over tighten.)
- 10. Install a clean or replacement diaphragm.
- 11. Put the large spring and slip ring into the bonnet, and screw the bonnet onto the regulator body.
- 12. Reinstall the regulator, mounting it securely to the main structure.
- 13. Reset the product regulator as described in Section 5.8.1.

2010-2229 Rev - B Page 21 of 52

5.9 High or Low Pressure Switch Replacement (Standard Unit only)

The high and low pressure alarms are activated by a pressure transducer located on the circuit board adjacent to the mains switch.

- Set the I/0 (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the back and front cabinets and the control panel.
 - 3. Disconnect the 7 pin connector from the circuit board.
- 4. Disconnect tubing from pressure sensor by cutting tie-wrap.
- 5. Remove the circuit board and replace with a new one.
- 6. Test the alarm system, as described in Section 2.3.
- 7. Reinstall the control panel and the front and back cabinets.

5.10 Circuit Breaker Replacement

5.10.1 Circuit Breaker Removal

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- Remove the cabinet front and back.
- Disconnect the circuit breaker leads.
- 4. Unscrew the circuit breaker retaining ring (front of control panel) while holding the circuit breaker body in place.
- 5. Remove the circuit breaker from the control panel.

5.10.2 Circuit Breaker Installation

Follow the removal procedure for the circuit breaker in reverse order to install the new circuit breaker.

5.11 I/0 (ON/OFF) Power Switch Replacement

5.11.1 I/0 (ON/OFF) Power Switch Removal

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position and unplug the power cord.
- Remove the cabinet front and back.
- 3. Unscrew the eight Phillips-head screws that hold the control panel to the main structure.
- 4. Disconnect the I/0 (ON/OFF) switch leads from the back of the switch being careful to note the color and position of each specific wire.
- 5. Push on the back of the power switch, while holding in its four retaining tabs, and remove the switch through the front of the control panel.

5.11.2 I/0 (ON/OFF) Power Switch Installation

Page 22 of 52 2010-2229 Rev - B

Follow the removal procedure for the I/O (ON/OFF) power switch in reverse order to install a new power switch.

5.12 Buzzer Replacement

The buzzer is a fixed component on the circuit board and is not individually replaceable.

5.13 Hour Meter Replacement

- 1. Set the I/0 (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- Remove the cabinet back.
- Disconnect the hour meter leads.
- 4. Push hour meter mounting tabs inward from the hour meter.
- 5. Remove the hour meter from the main structure.
- 6. Install the new hour meter into its mounting location, and push the mounting tabs outward to secure hour meter. Make sure that the hour meter is mounted right side up.
- 7. Reconnect the hour meter leads.
- Reinstall the cabinet back.

5.14 Flow Meter Replacement

5.14.1 Flow Meter Removal

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the front and back cabinets, as well as the control panel. Ref. 5.2.1 thru 5.2.3. On units with OCSI, it is necessary to also first remove the circuit board.
- 3. Remove the 3/8-inch silicone oxygen tubing from the flow meter fittings.
- 4. Unscrew the flow meter nuts with a wrench or deep socket.
- 5. Remove the flow meter through the control panel.

5.14.2 Flow Meter Installation

To install a new flow meter, follow the flow meter removal procedure in reverse order. Then perform a leak test on the connections.

5.15 Power Cord Replacement

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position and unplug the power cord.
- 2. Remove the cabinet back.
- 3. Slide the power cord strain relief reinforcement upwards to remove it from the mounting location at the rear of the base.
- 4. Disconnect the power cord leads from the terminal quick connects.

- 5. Connect the leads on the new power cord at the terminal quick connects.
- 6. Reinstall the power cord strain relief into the base of the unit.
- 7. Reconnect the cabinet back.

6.0 Troubleshooting

6.1 Air Pressure Test (P1)

Testing the operating pressure is a useful diagnostic tool when a concentrator has low purity and requires servicing. Units functioning normally do not require operating tests.

Use the following procedure to test the operating pressure of the unit.

- 1. Set the I/0 (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the cabinet rear.
- 3. Remove the air supply tubing going to the control valve and install the test port T-fitting. Figure 6.1.1 shows the normal operating configuration for the unit. Figure 6.1.2 shows the installation of the test ports.
- 4. Connect the pressure test gauge to the test port.
- 5. Plug in the power cord, and set the I/O (ON/OFF) power switch to the ON position. Set the flow meter to 8 I/min, and allow the unit to run at least five minutes.
- 6. Observe the maximum and minimum readings on the pressure test gauge.
- 7. The maximum reading should not exceed 40 psig (275 kPa). The minimum reading should not be less than 13 psig (90 kPa).

NOTE When you turn the unit on, it will take several minutes to reach normal operating pressures.

Page 24 of 52 2010-2229 Rev - B

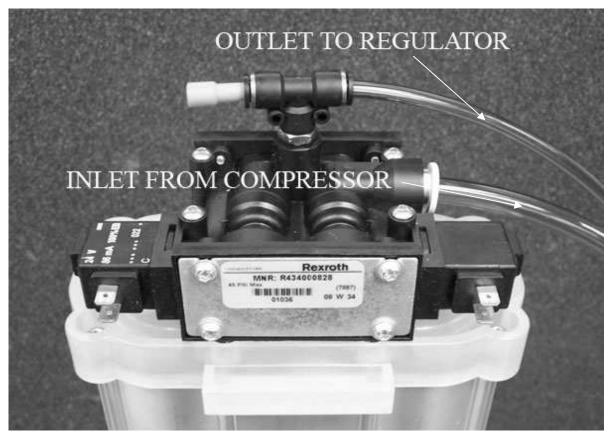


Fig. 6.1.1

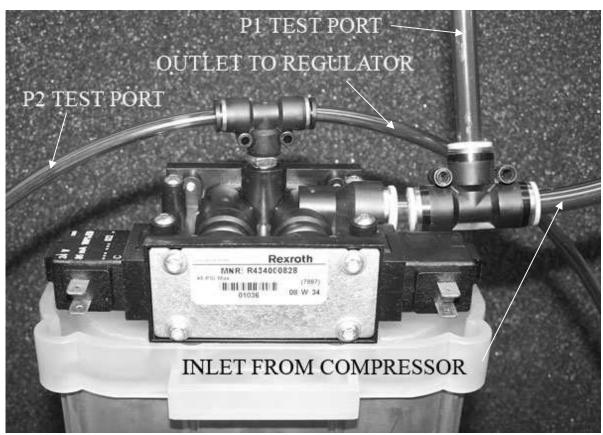


Fig 6.1.2

2010-2229 Rev - B Page 25 of 52

6.1.1 High Operating Air Pressure (P1)

Higher than normal operating pressure may indicate any of the following:

- A restrictive exhaust muffler, which does not allow the waste (purge) gas to exit the system freely.
- Contaminated sieve beds. Change the sieve beds.

6.1.2 Low Operating Air Pressure (P1)

Lower than normal operating pressure may indicate any of the following:

- A restriction in the suction resonator or air intake filter, this limits the amount of room air available to the compressor. Disconnect the suction tube at the compressor, and allow the unit to operate without the suction resonator to see if normal operating pressure returns.
- An improperly operating control valve. Confirm that the control valve does not have a leak.
- A leak in the unit, which allows system pressure to escape. Leak test the unit.
- A compressor with reduced output.

Ensure that the concentration level at the desired liter flow is within specifications listed in section 2.4. If it is below specifications, replace or repair the compressor.

6.2 Product Pressure Test (P2)

Testing the product pressure is a useful diagnostic tool when a concentrator has low purity and requires servicing. Units functioning normally do not require operating tests.

Use the following procedure to test the product pressure of the unit.

- 1. Set the I/O (ON/OFF) switch to the 0 (OFF) position, and unplug the power cord.
- 2. Remove the cabinet rear.
- 3. Remove the plug from the T-fitting atop the valve and install the test port tubing (1/4" OD tubing is required). Figure 6.1.1 shows the normal operating configuration for each unit. Figure 6.1.2 shows the installation of the test port tubing.
- 4. Connect a pressure test gauge to the P2 test port tubing.
- 5. Plug in the power cord, and set the I/O (ON/OFF) power switch to the ON position. Set the flow meter to 8 I/min, and allow the unit to run at least five minutes.

Page 26 of 52 2010-2229 Rev - B

6. Observe the maximum and minimum readings on the pressure test gauge. The maximum reading should not exceed 40 psig (275kPa). The minimum reading should not be less than 13 psig (90 kPa).

6.2.1 Low Product Pressure (P2)

Lower than normal operating pressure may indicate any of the following:

- An inlet air filter that limits the amount of room air available to the compressor. Disconnect the suction tube at the compressor, and allow the unit to operate without the suction resonator to see if normal operating pressure returns.
- An improperly operating control valve. Confirm that the control valve does not have a leak.
- A leak in the unit, which allows system pressure to escape. Leak test the unit.
- A compressor with reduced output. Ensure that the concentration level at the desired liter flow is within specifications listed in Section 2.4. If it is below specification, replace or repair the compressor.

6.2.2 High Product Pressure (P2)

Higher than normal operating pressure may indicate any of the following:

- A restrictive exhaust muffler, which does not allow the waste (purge) gas to exit the system freely.
- Check exhaust muffler for any restrictions
- Contaminated sieve beds. Change the sieve beds.

6.3 General Troubleshooting

Before reviewing the troubleshooting chart, the following steps may be useful to isolate any malfunctions:


- 1. Turn the concentrator on. If unit does not turn on, refer to troubleshooting chart.
- 2. Make sure all filters are clean.
- 3. Turn flow meter completely open, up to the highest setting. The ball should rest at the top of the scale. If setting is not greater than 8.5 l/min, adjust regulator so that the ball level exceeds 8.5 l/min with the flow meter completely open.
- 4. Connect test pressure gauge to the outlet fitting of the unit. The pressure should read approximately 17 psig (117 kPa).

2010-2229 Rev - B Page 27 of 52

- 5. Connect test pressure gauge to the P1 test port on the valve inlet. The pressure should be cycling between approximately 13 and 40 psig (90 and 275 kPa).
- 6. Make sure the unit is cycling properly by observing the pressure gauge cycle between a high and a low pressure. If the unit is not cycling properly, refer to troubleshooting chart.
- 7. Make sure that the unit is leak free by testing all tubing connections and fittings with leak testing solution. Protect circuit board from solution and start leak test at the compressor outlet, following the air flow through the unit to the oxygen outlet. Repair all leaks by tightening connections and fittings.
- 8. Set the concentrator at 8 l/min and connect pressure test gauge to P2 at the top of the sieve module. Determine pressure parameters by observing high and low pressure points on the gauge. It should cycle between approximately 13 psig and 40 psig (90 to 275 kPa). If pressures are high or low, refer to the troubleshooting chart.
- 9. Review troubleshooting chart to isolate and repair any other malfunctions.

The following diagnostic flow chart will help to isolate potential problems.

Page 28 of 52 2010-2229 Rev - B

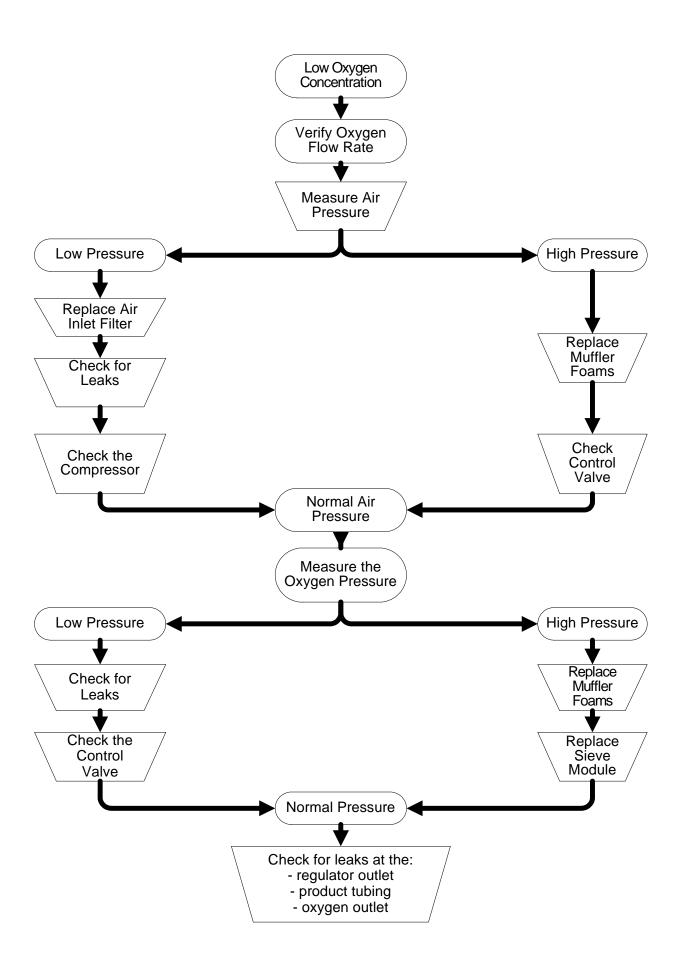

6.4 Troubleshooting Chart

Problem	Probable Cause	Solution
Compressor does not run.	No power to unit.	Check wall outlet for power.
Constant audible alarm with I/0 (ON/OFF) power switch in ON position.	Unit circuit breaker tripped or faulty.	Reset or replace circuit breaker.
	Faulty electrical connections. Defective circuit board.	Check electrical connections.
	Defective I/0 (ON/OFF) power switch.	Check power switch. Replace I/0 (ON/OFF) power switch.
Compressor runs with intermittent alarm. Concentration at 8 L/MIN is within specifications.	Defective high/low pressure switch. Kinked tubing.	Replace and retest pressure switch, if a standard board is installed.
Compressor shuts down intermittently,	Restricted air flow through unit.	Clean inlet filter, or remove obstruction.
	Unit overheating due to improper location.	Locate unit away from heating source, providing adequate ventilation on all sides.
	Defective cooling fan.	Replace fan.
Compressor does not start. I/0 (ON/OFF) power switch in ON position, intermittent	Extreme cold start.	Allow unit to reach room temperature.
in ON position, intermittent alarm, and cabinet fan turns.	Compressor thermally cut off due to excessive heat. NOTE: It will not restart unit it cools down,	Blocked air intake or defective cooling fan. Clear obstruction, or replace cooling fan.
	Defective capacitor.	Replace capacitor.
	Faulty electrical connection for compressor.	Check electrical connections for compressor.
Compressor runs with intermittent low pressure alarm and low oxygen concentration.	Leak	Leak test and repair leak.

Page 30 of 52 2010-2229 Rev - B

Problem	Probable Cause	Solution
Compressor runs with	Defective sieve module.	Replace sieve module.
intermittent high pressure		
alarm and low oxygen	Restriction in exhaust	Replace or clean muffler
concentration.	muffler.	foam.
	Defective valve.	Replace sieve module.
Compressor relief valve	Defective control valve.	Replace control valve.
releases (popping sound).	Contominated signs	Danlaga siaya madula
	Contaminated sieve module.	Replace sieve module.
	module.	Replace relief valve.
	Defective relief valve.	Replace Teller valve.
Constant alarm with I/0	Defective circuit breaker.	Replace circuit breaker.
(ON/OFF) switch in ON		
position. Circuit breaker	Defective capacitor.	Replace capacitor.
repeatedly trips.		·
	Defective compressor.	Replace compressor.
	Defective circuit board.	Replace circuit board.
	Faulty electrical connection.	Repair electrical connection.
Alarm does not sound.	Dead battery.	Replace battery.
Alaim does not sound.	Dead battery.	Replace battery.
	Incorrectly installed battery.	Reinstall battery with correct
		polarity.
	Faulty electrical connection.	Repair electrical connection.
	Defective I/0 (ON/OFF)	Replace I/0 (ON/OFF) switch.
	switch.	
	Defective buzzer.	Replace control board.
	Defective buzzer.	Poplace and test central
	Defective pressure sensor.	Replace and test control board (Standard unit only).
Flow meter fluctuates.	Improperly set or faulty	Check regulator setting/clean,
now meter naticates.	product regulator.	repair, or replace regulator.
	F. 13331.0931.41011	
		Test for leaks.
	Leak.	
		Replace compressor
	Worn compressor.	
	Defeative flavores to	Replace flow meter.
	Defective flow meter.	Chook tubing that connects
	Kinkod tubing	Check tubing that connects
	Kinked tubing	the top of the sieve beds.

2010-2229 Rev - B Page 31 of 52

Problem	Probable Cause	Solution
Cooling fan does not turn.	Defective cooling fan.	Replace cooling fan.
	Defective electrical connections.	Check electrical connections.
Limited or low flow.	Restriction in humidifier or tubing.	Replace humidifier or tubing.
	Product regulator set too low.	Adjust regulator setting.
	Leak.	Leak test and repair leak.
	Weak compressor.	Check system pressure, and rebuild or exchange compressor.
	Air flow obstruction.	Check filter, suction resonator, and suction tube for obstruction.
Low concentration.	Inlet air filter is dirty or partially blocked.	Replace inlet air filter.
	System leak	Leak test and repair leak.
	Faulty compressor	Check system pressure, and rebuild or replace compressor.
	Unit temperature too high,	Blocked air intake or dirty inlet air filter, faulty cooling fan.
	Contaminated sieve module.	Check that P1 and P2 pressures are within range. Replace sieve module.
	Defective control valve.	Repair or replace sieve module.
	Restriction in exhaust muffler,	Replace or clean exhaust muffler.
	Restriction of inlet air.	Check inlet tubing for obstruction and remove.

Page 32 of 52 2010-2229 Rev - B

6.5 Tool Kit and Pressure Test Gauge

The tools needed for you to properly service the Nuvo 8 unit are listed below:

- Requires no special tools; generally available tools including common pliers, channel lock, wire cutters, needle-nose pliers, slotted-head screwdriver, long Phillips head screwdriver, 8-inch adjustable wrench, 7/16-inch socket, 7/16-inch combination wrench, 5/8-inch combination wrench and 3/8-inch combination wrench.
- For convenience, it is desirable to have a battery operated drill motor available to remove and install screws.
- An accurate pressure test gauge to take both high and low pressure readings on the Nuvo 8 unit should be kept available at all times. This gauge connects to the pressure test ports on top of the air tank and on top of the sieve bed.

Appendices

Drawings


Pneumatic Flow Chart	A-1
Electrical Diagram 230 Volt	A-2
Electrical Diagram 115 Volt	A-3

Exploded Drawings

Base Assembly	A-4
Main Structure Assembly (front)	A-5
Main Structure Assembly (rear)	A-5.1
Pressure Regulator Assembly	A-6
Compressor Assembly	A-7
Control Panel Assembly (front)	A-8
Control Panel Assembly (rear)	A-8.1
Front Cabinet Assembly	A-9
Rear Cabinet Assembly	A-10
Sieve Module	A-11
Maintenance Log	A-12


Refer to Thomas Compressor Manual (FSM 0159 Rev A dtd 07/04) for compressor rebuild procedures. Call Nidek Medical customer service to obtain a pdf file copy.

2010-2229 Rev - B Page 33 of 52

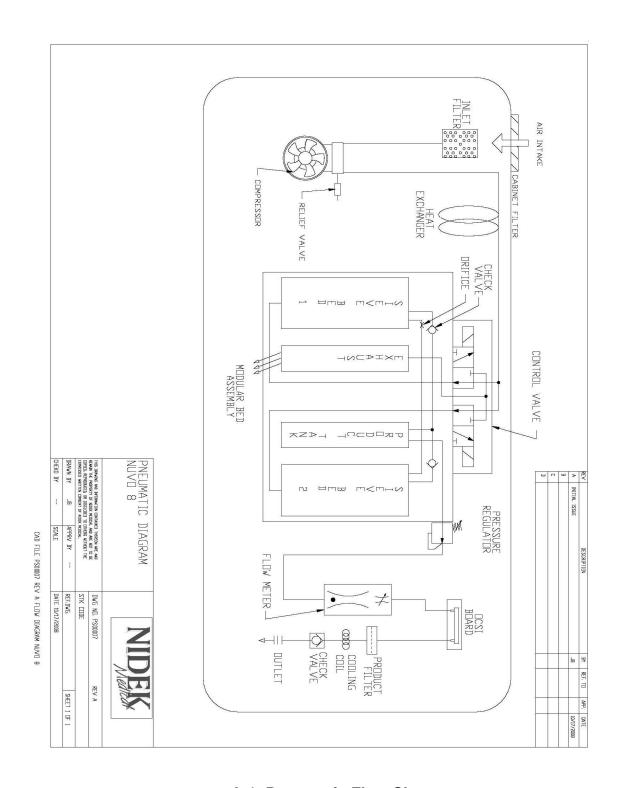


A-1, Pneumatic Flow Chart PS00007, with OCSI

Page 34 of 52 2010-2229 Rev - B

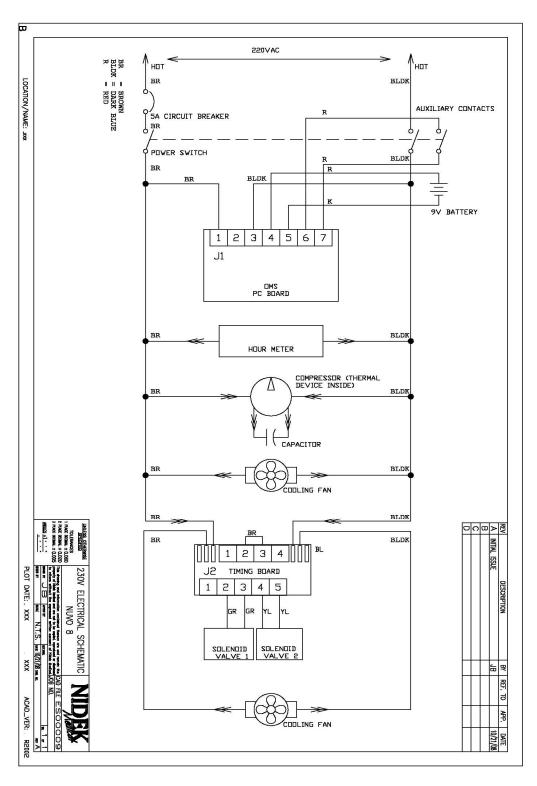


A-2, 230 Volt Electrical Schematic ES00009, with OCSI

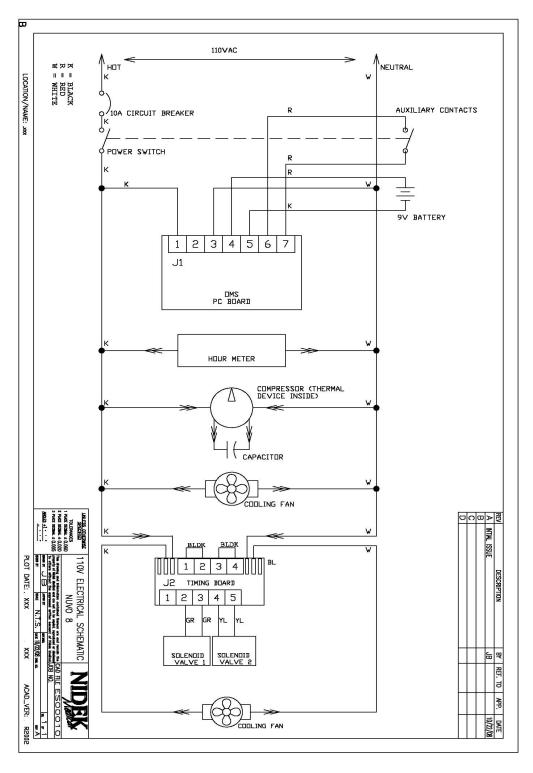

A-3, 115 Volt Electrical Schematic ES00010, with OCSI

Page 36 of 52 2010-2229 Rev - B

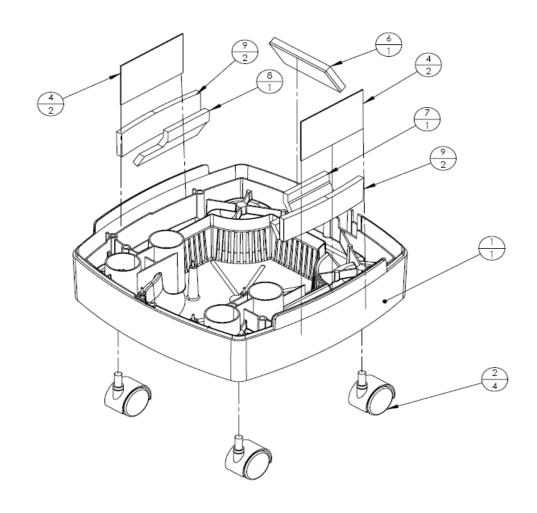


A-4, BASE ASSEMBLY

0100-0380 REV A

Part Reference			Qty	Unit of
Number	Number	Part Description	Required	Measure
9251-1502	2 1	BASE, MODIFIED FOR 8LPM UN	IT 1	1.000 EA
8300-8068	3 2	CASTER	4	4.000 EA
9251-1309	4	ISODAMP,BASE,LF/RT INSIDE	2	2.000 EA
9251-1315	6	FOAM,BASE FAN REAR INSIDE \	VALL 2	2.000 EA
9251-1316	5 7	FOAM,BASE RT. WALL INSIDE	•	1.000 EA
9251-1317	8	FOAM,BASE LF. WALL INSIDE	•	1.000 EA
9251-1318	9	FOAMBASE L/R OUTSIDE WALL		2.000 EA

2010-2229 Rev - B Page 37 of 52

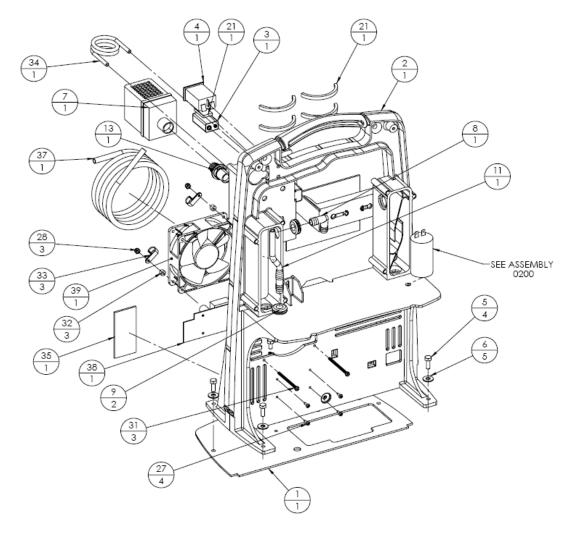


A-5, MAIN STRUCTURE ASSEMBLY FRONT VIEW

NOTE: MAIN STRUCTURES ARE IDENTICAL, EXCEPT FOR VOLTAGE SPECIFIC COMPONENTS.

Use Assembly #0140-0380 for the 115 volt unit Use Assembly #0140-0381 for the 230 volt unit

Page 38 of 52 2010-2229 Rev - B

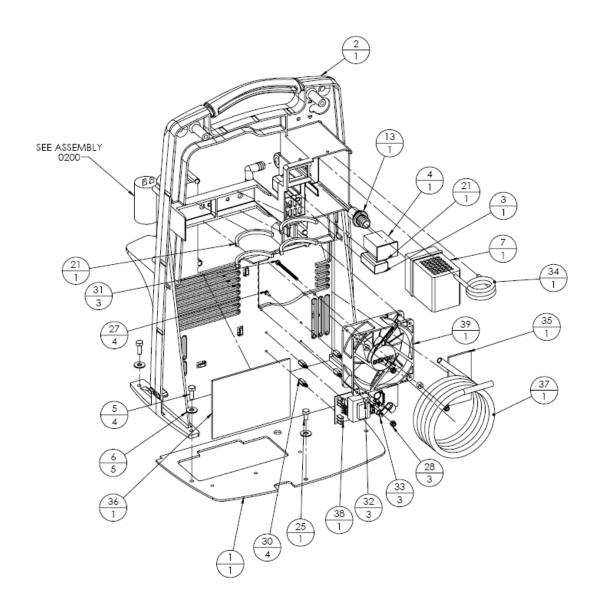

A-5.1, MAIN STRUCTURE ASSEMBLY REAR VIEW

NOTE: MAIN STRUCTURES ARE IDENTICAL, EXCEPT FOR VOLTAGE SPECIFIC COMPONENTS.


Use Assembly #0140-0380 for the 115 volt unit Use Assembly #0140-0381 for the 230 volt unit

2010-2229 Rev - B Page 39 of 52

A-5-1, MAIN STRUCTURE ASSEMBLY PARTS

Part Reference	Call-Out		Qty Unit of
Number	Number	Part Description F	Required Measure
9251-1501	1	BASE, PLATE MODIFIED	1.000 EA
9251-1503	2	STRUCTURE,MAIN NUVO	1.000 EA
9250-1004	1	STRUCTURE,MAIN NUVO	1.000 EA
9251-1402	3	BATTERY, DURACELL 9 VOLT	1.000 EA
8400-5018	4	HOURMETER 100-230V 50/60~	1.000 EA
9250-1047	5	BOLT, 1/4 -20 X 3/4" LG	4.000 EA
9250-1020	6	WASHER,FLAT 1/4"	5.000 EA
9250-1180	7	FILTER, AIR INLET	1.000 EA
6814-9228	8	ELBOW, 1/2 DOUBLE BARB	1.000 EA
9250-1030	9	GOMMET,RUBBER	2.000 EA
9250-1069A	. 11	TUBE,ASSY.SOUND INLET	1.000 EA
9251-9226	1	CONNECTOR, NYLON 1/2 HOSE	1.000 EA
9250-1069	2	TUBE,.25"IDX,375"ODX4.25	1.000 EA
9251-1508	12	HARNESS,WIRING 230 VOLT	1.000 EA
9250-1059	13	CONNECTOR, MOLDED INLET	1.000 EA
9251-1533	19	TIE-WRAP, 24" LONG	1.000 EA
9250-1102	21	FOAM,KIT VIBRATION	1.000 EA
9030-6008	22	TIE-WRAP 4.5" OVERALL	13.000 EA
9250-1058	25	BOLT, 1/4-20X 5" LG.GRADE2	1.000 EA
5190-2233	26	HOLDER TIE WRAP	3.000 EA
6120-9854	27	SCREW #6 TYPE ~A~ X 1/2, SHEE	T 4.000 EA
9200-0502	28	NUT,8-32 NYLON LOCKING	4.000 EA
9250-1510	29	HARNESS,WIRING 220 VOLT	1.000 EA
9251-1516	30	STANDOFF,PLASTIC	4.000 EA
9251-1520	31	SCREW, 8-32X 2.5" PHILLIPS	3.000 EA
9251-1521	32	SPACER,3/8"ODX.175"IDX1/4"LG	3.000 EA
9251-1522	33	CLAMP, 0.375" TUBING	3.000 EA
9251-1523	34	EXCHANGER, HEAT 8 LPM	1.000 EA
9251-1524	35	FOAM VENT COVER LEFT	1.000 EA
9251-1525	36	FOAM VENT COVER RIGHT	1.000 EA
9251-1594	37	EXCHANGER,HEAT 3/8" O.D.8 LP	M 1.000 EA
8400-1209	38	BOARD,TIMING 8 LPM	1.000 EA
9250-1024	39	FAN, 230V LOW NOISE	1.000 EA
9250-1026	99	CLAMP, HOSE NYLON.67"78"I	D 1.000 EA


Page 40 of 52 2010-2229 Rev - B

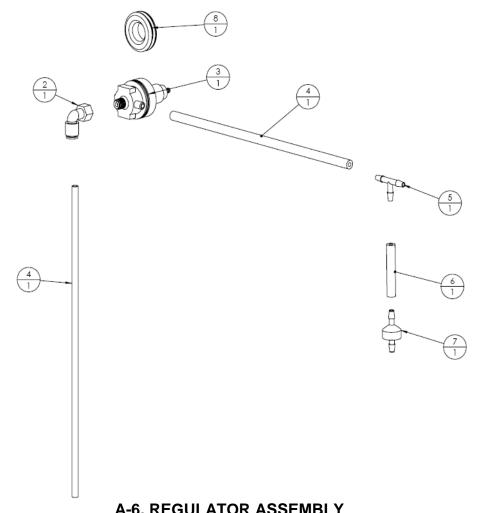

A-0, REGULATOR ASSEMBLT	

Part Reference Number	Call-Out Number	Part Description	Qty Required	Unit of Measure
7854-6019	1	HOSE, 1/4 OD X 10.5 LG	1.0	00 EA
6814-9221	2	ELBOW, 1/8 NPTF X 1/4 ODT	1.0	00 EA
8400-1060	3	REGULATOR, 2-PORT	1.0	00 EA
7854-6054	4	HOSE 5/32 X 11/32 X 10"LG SIL	1.0	00 EA
6491-1008	5	TEE NYLON 3/16 HOSE BARB	1.0	00 EA
7854-6052	6	HOSE 5/32 X 11/32 X 2 LG SILIC	1.0	00 EA
6956-9674	7	VALVE, CHECK 1/4 HOSE MPC A9	75 1.0	00 EA

2010-2229 Rev - B Page 41 of 52

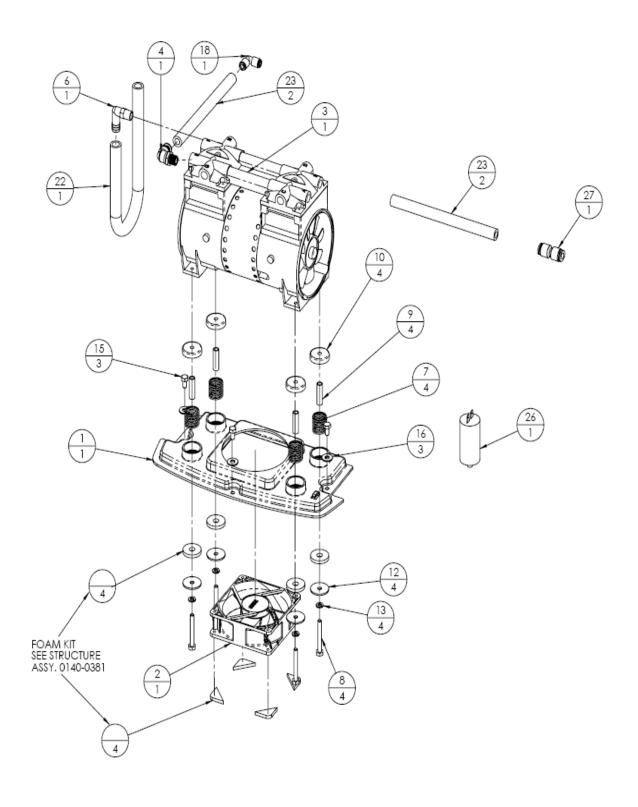


A-7, COMPRESSOR ASSEMBLY

Page 42 of 52 2010-2229 Rev - B

A-7-1, COMPRESSOR ASSEMBLY PARTS

Part Reference				Unit of
Number	Number	Part Description Required		Measure
0050 4040	4	DI ATE COMPRESSOR	1.00	00 = 4
9250-1018	1	PLATE, COMPRESSOR	1.00	
9250-1023*	2	FAN, 115V LOW NOISE	1.00	
9250-1024	2	FAN, 230V LOW NOISE	1.00	_, .
9250-1515*	3	COMPR, 115/60 NEXUS 8 LPM	1.0	00 EA
9251-1532	3	COMPR, 230/50 NEXUS 8 LPM	1.00	00 EA
9251-1052	4	FITTING,PLASTIC1/4"MNPT3/8"TU	B 1.00	00 EA
6191-5003	6	ELBOW, NYL 1/4 NPT X 1/2 HOSE	1.00	00 EA
9250-1016	7	SPRING,COMPRESSOR VIBRATIO	N 4.00	00 EA
9250-1105	8	BOLT, 1/4"X20X2"LGGRADE 2	4.00	00 EA
9250-1111	9	SLEEVE, .3/8" O.D. X 1.45" LG	4.00	00 EA
9250-1112	10	CAP, COMPRESSOR SPRING TOP	4.00	00 EA
9250-1109	12	WASHER,FLAT .25"ID X 1.25"OD	4.00	00 EA
9250-1110	13	WASHER,LOCK 1/4" ID.	4.00	00 EA
9250-1058	15	BOLT, 1/4-20X 5" LG.GRADE2	3.00	00 EA
9250-1020	16	WASHER,FLAT 1/4"	3.00	00 EA
9250-1163	18	FITTING, 3/8" TUBE ELBOW	1.00	00 EA
9030-6008	21	TIE-WRAP 4.5" OVERALL	1.00	00 EA
8400-1028	22	HOSE, COMPRESSOR INLET.	1.00	00 EA
9251-1531	23	TUBING,3/8"ODX 8"LG	2.00	00 EA
9251-1517	24	SPACER, SPRING	4.00	00 EA
5161-1017	25	TERMINAL, FEMALE INSULATED	2.00	00 EA
9250-1322	26	CAPACITOR,15UF W/STUD MOUN	T 1.00	00 EA
9251-1541	27	FITTING,STRAIGHT3/8" Q.D.	1.00	00 EA

2010-2229 Rev - B Page 43 of 52

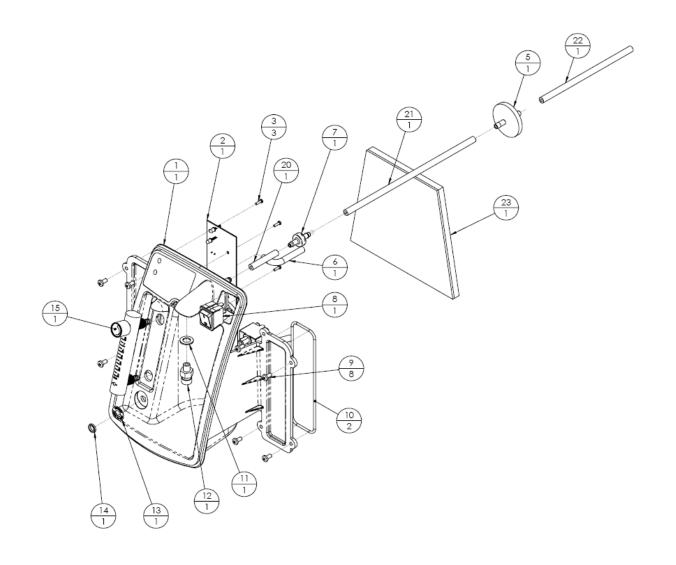


A-8 CONTROL PANEL ASSEMBLY UNIT WITH OCSI FRONT VIEW

NOTE: CONTROL PANELS ARE IDENTICAL, EXCEPT FOR VOLTAGE SPECIFIC COMPONENTS.

Use Assembly #0300-0380 for the 115 volt unit Use Assembly #0300-0385 for the 230 volt unit

Page 44 of 52 2010-2229 Rev - B

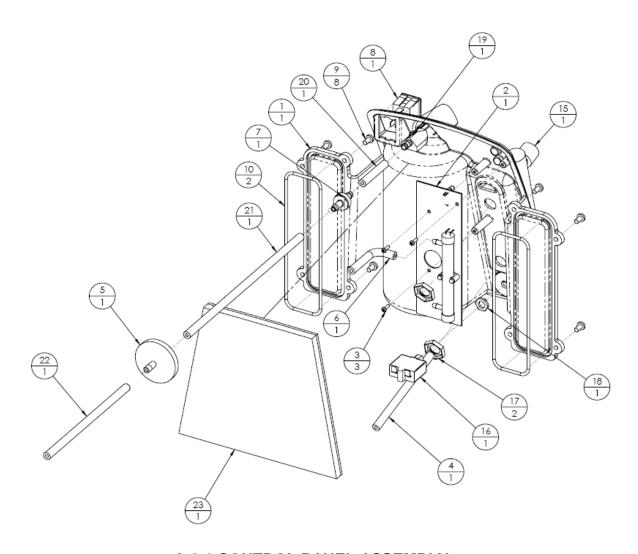

A-8.1 CONTROL PANEL ASSEMBLY
UNIT WITH OCSI
REAR VIEW

NOTE: CONTROL PANELS ARE IDENTICAL, EXCEPT FOR VOLTAGE SPECIFIC COMPONENTS.

Use Assembly #0300-0381 for the 115 volt unit Use Assembly #0300-0385 for the 230 volt unit


2010-2229 Rev - B Page 45 of 52

A-8-1 CONTROL PANEL ASSEMBLY UNIT WITH OCSI

* Denotes 115V Components


Part Reference			Qty	Unit of
Number	Number	Part Description	Required	Measure
9250-1005	1	PANEL,CONTROL	1.0	00 EA
9250-1080	2	BOARD,OMS UNV	1.0	00 EA
9250-1380	2	BOARD,OMS COMPASS	1.0	00 EA
9250-1045	3	SCREW,PLASTITE#4X3/8" PAN.HI	3.0	00 EA
9250-1041	4	HOSE 5/32 X 11/32 X 5 LONG SIL	1.0	00 EA
7631-1053	5	FILTER, BACTERIAL	1.0	00 EA
7854-6051	6	HOSE 5/32 X 11/32 X 3 LG SILIC	1.0	00 EA
6956-9674	7	VALVE, CHECK 1/4 HOSE MPC A9	75 1.0	00 EA
9250-1012*	8	SWITCH,POWER 115 VOLT	1.0	000 EA
9250-1013	8	SWITCH,POWER 220 VOLT	1.0	00 EA
9250-1050	9	SCREW, 10-32 X 3/8 PAN HD.PHIL	8.0	00 EA
9250-1040	10	O-RING, 5.0"ID X .125 THICK	2.0	00 EA
9250-1048	11	WASHER, NYLON .44"IDX.75"X.062	2" 1.0	00 EA
9251-1122	12	HUMIDIFIER FITTING DISS	1.0	00 EA
9251-1526	15	FLOWMETER, 2-8.0 LPM	1.0	00 EA
8400-1018*	16	BREAKER,CIRCUIT 10 AMP	1.0	00 EA
8400-1019	16	BREAKER,CIRCUIT 5 AMP	1.0	00 EA
9250-1021	19	HUMIDIFIER BLOCK	1.0	00 EA
7854-6052	20	HOSE 5/32 X 11/32 X 2 LG SILIC	1.0	00 EA
7854-6054	21	HOSE 5/32 X 11/32 X 10"LG SIL	1.0	00 EA
7854-6055	22	HOSE 5/32 X 11/32 X 7"LG SIL	2.0	00 EA

Page 46 of 52 2010-2229 Rev - B

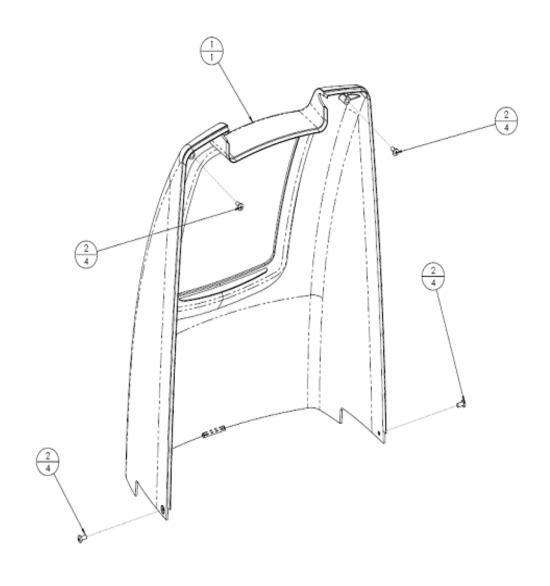

A-9, FRONT CABINET ASSEMBLY 0500-0380 Rev. A


Part Reference Number	Call-O Numb		Part Description	Qty Required	Unit of Measure
9250-1002	1	CAE	BINET,FRONT NUVO	1.000	EA
9250-1050	2	SCF	REW, 10-32 X 3/8 PAN HD.PHIL	4.000	EA

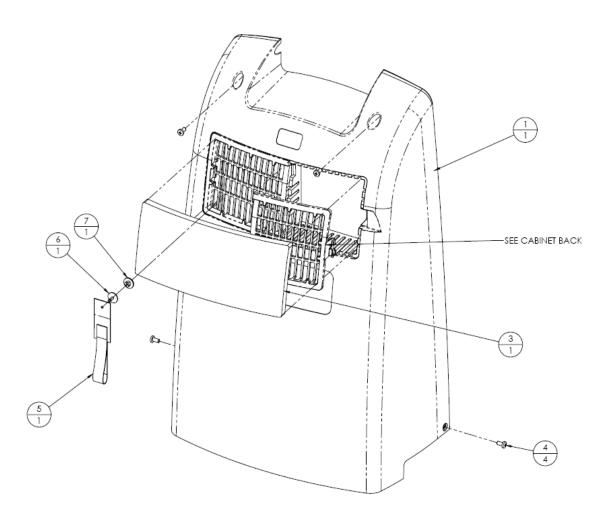

A-10, REAR CABINET ASSEMBLY
NUVO 0550-0380 REV A

Part Reference Number	Call-Out Number	Part Description	Qty Required	Unit of Measure
		·	•	
9250-1200	1	CABINET,BACK W/ACCESS DOOR	1.000	EA
9250-1025	3	FILTER,CABINET INLET	1.000	EA
9250-1050	4	SCREW, 10-32 X 3/8 PAN HD.PHIL	4.000	EA
8400-0022	5	RETAINER,CORD VELCRO	1.000	EA
8400-0023	5	RETAINER,CORD RIVET	1.000	EA
9114-0G13	5	CORD RETAINER	1.000	EA
6490-9922	6	SNAP RIVET, NYLON 0.320/0.340	1.000	EA
6490-9925	7	SNAP RIVET, NYLON FEMALE	1.000	EA

Page 48 of 52 2010-2229 Rev - B

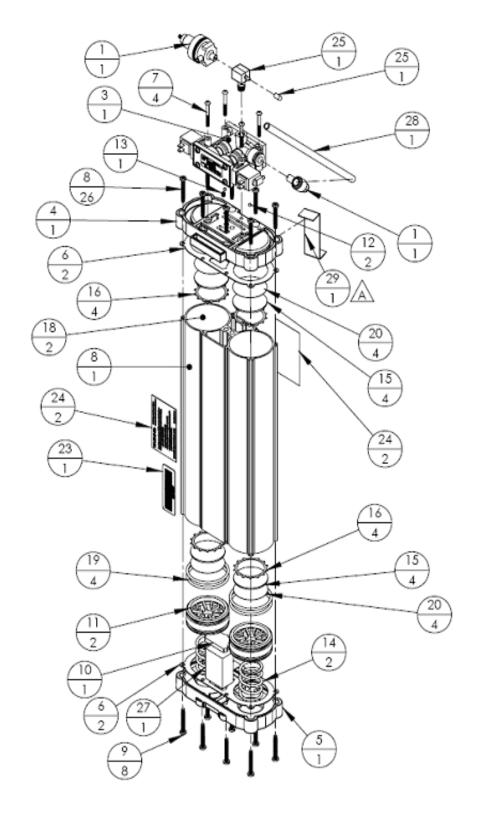

A-11, SIEVE MODULE ASSEMBLY 0600-0480

2010-2229 Rev - B Page 49 of 52

A-11-1, SIEVE MODULE ASSEMBLY PARTS

0600-0480

Part Reference Number	Call-Out Number	Part Description	Qty Required	Unit of Measure
Number	Number	Fait Description	required	Weasure
8400-1200	3	VALVE ASSY, REXROTH	1.000	EA
8400-1226	4	PLATE TOP-CROWNED	1.000	EA
8400-1227	5	PLATE,BOTTOM- CROWNED	1.000	EA
8400-1212	6	GASKET	2.000	EA
8400-1213	7	SCREW,8-32 X 1 .5" LG.	4.000	EA
8400-1260	8	TUBE, SIEVE EXTRUDED 16"LG.	1.000	EA
8400-1265	8	TUBE, SIEVE EXTRUDED 16"LG.	1.000	EA
8400-1221	9	SCREW, #10-24X1.25 TYPE F	8.000	EA
8400-1322	10	MUFFLER, FELT NEW	1.000	EA
8400-1230	11	PISTON	2.000	EA
8400-1234	12	VALVE, CHECK INSERT	2.000	EA
8400-1251	13	ORIFICE 2.5MM 475LOMN	1.000	EA
8400-1233	14	SPRING	2.000	EA
8400-1350	15	FILTER, 2"OD POLY 25MICRON	4.000	EA
6490-9926	16	KEEPER, 2- WALDES	4.000	EA
8615-4044	18	SIEVE, ZEOX OXY2	1.240	EA
9251-1350	20	FILTER, 2"OD.304SS MESH	4.000	EA
9010-5G18	23	LABEL SIEVE BED BAR CODE	1.000	EA
9010-5G16	24	LABEL WARNING DECAL	2.000	EA
8644-9401	30	PLUG, 1/4~ ODT PUSH IN	1.000	EA
8400-1051	98	FOAM, EXHAUST MUFFLER	1.000	EA
9250-1051	98	FOAM, EXHAUST MUFFLER	1.000	EA
6816-4162	99	TEE, MALE BRANCH, 1/8 NPTM X 1	1.000	EA
8400-1221	99	SCREW, #10-24X1.25 TYPE F	16.000) EA
8400-1225	99	SCREW 10-32x1.5"LG TORX	16.000) EA
8400-5013	99	REDUCER INSERT, 5/16"-1/4" EL	1.000	EA
8644-9401	99	PLUG, 1/4~ ODT PUSH IN	1.000	EA
9251-1228	99	TUBING,3/8"x10"LG.	1.000	EA
9251-1534	99	BRACKET,SIEVE BED	1.000	EA
8400-1351	99999	FILTER, 2-5"OD POLY 25MICRON	2.000	EA

Page 50 of 52 2010-2229 Rev - B

Nidek Medical Oxygen Concentrator Service and Maintenance Log

	1/14/11/01/41/05
Model Number	Serial Number

Initial Inspection

- 1. Upon receipt, check the unit for shipping damage. Notify shipping company if damaged.
- 2. Verify that cabinet air filter and the inlet air filter are in place.
- 3. Plug the unit into an electrical outlet, turn the unit 'ON,' and check the audible/visual alarms.
- 4. Set the flow meter/flow control at the maximum recommended flow rate and allow the unit to run for 15 minutes.
- 5. Using a calibrated oxygen analyzer, verify concentration is greater than 87 percent.

Routine Service Check

Perform routine servicing as shown in the chart below. Record the activities performed in the log provided on the following page.

- 1. Record the elapsed usage time in hours.
- 2. Check oxygen concentration with a calibrated oxygen analyzer.
- 3. Verify audible alarm and indicator light functions between patients and every two years.
- 4. Inspect filters and replace as necessary.

Between-Patient Maintenance

- 1. Remove oxygen tubing, cannula, and humidifier bottle and discard.
- 2. Wash or replace the humidifier tubing if used.
- 3. Wash or replace the cabinet air filter.
- 4. Clean the concentrator cabinet.
- 5. Check oxygen concentration and flow. If the unit performs within specification, the final product filter does not need to be replaced between patients.

Patient/Caregiver Maintenance

- 1. Inspect the Oxygen tubing, cannula, and humidifier bottle clean as needed according to manufacturer's instructions.
- 2. Wash the cabinet air filter weekly with a mild detergent solution. Dry before reinstalling onto the device.

The routine service intervals shown below depend on the conditions in which the devices are used. They reflect the **minimum recommendation** when operated in a clean environment. As conditions can vary widely, the homecare provider or patient caregiver is responsible to determine:

- the character of the environment in which the concentrator is to operate.
- a maintenance schedule with intervals based on the environment in which the unit is operating/functioning.

Standard Servicing Intervals are shown below. Intervals used by the homecare service provider and/or patient caregiver should be more frequent when conditions of usage dictate.

Nidek Medical Oxygen Concentrator Routine Service Intervals					
Check Percent					
Oxygen	Cabinet Air Filter	Inlet Air Filter	Final Product Filter	Capacitor	
Concentration					
OCSI Models: Every	Wash the filter	Inspect at each			
15,000 hours	each week in a	patient visit. Replace	Replace at each	Replace at each	
or 3 years.	mild detergent	annually, or more	compressor	compressor	
Std Models: Every	solution.	often depending on	service / module	service / module	
5,000 hours or 1	Dry before	environment.	replacement.	replacement.	
year.	reinstalling.				

Please maintain a log of all maintenance activities performed on this unit.

Serial Number	Model

Date	Hours	% O2	Alarms Check	Additional Information (Work Done, Filter Changes, Comments, etc)					
Inspection Prior to Putting Into Service									
		_							
In-Service Checks									

Medical device regulations require users and service personnel to notify manufacturers of any incidents that, if repeated, could cause injury to any person.

email: info@nidekmedical.com

Please update maintenance log information upon each service at www.nidekmedical.com under the 'Maintenance Log' tab.